

gelFAST™ anywear

38.6 mL / 1.3 fl oz


INSTRUCTIONS




lanyard: gelFAST may be worn on a necklace, while not interfering with ID badges, fobs and keys.

garment clip: allows users to wear gelFAST on a wide array of clothing, including pants, skirts, scrubs, jacket pockets, knap-sacks, purses, etc.


USAGE INSTRUCTIONS

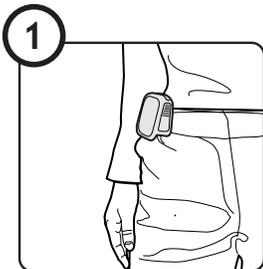

1
gelFAST Anywear may be worn using integral garment clip

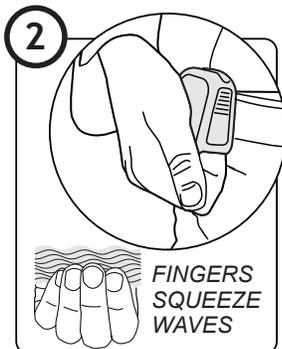


2
Squeeze "waves" on cartridge (bottle) with fingertips to dispense gel.

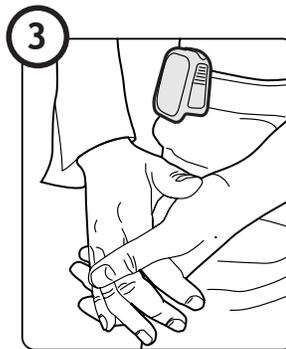

3
Rinse hands until dry (about 15 sec.)

CARTRIDGE REMOVAL

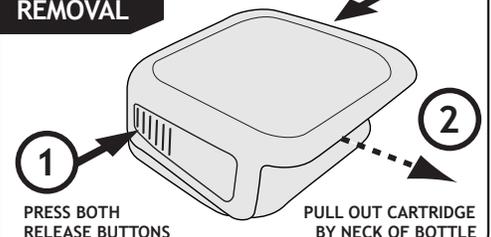

1
PRESS BOTH RELEASE BUTTONS



2
PULL OUT CARTRIDGE BY NECK OF BOTTLE

CARTRIDGE INSERTION

HOLSTER

REFILL CARTRIDGE

REMOVE LINER

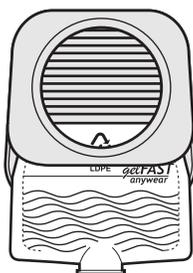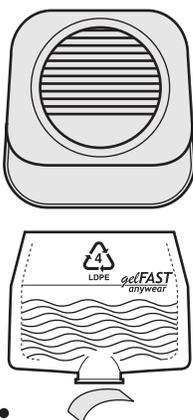

> CLICK <

Designed & Made by:
Medonyx Inc.

For more information, please visit us at:
www.medonyx.com

1 (866) MEDONYX (1-866-633-6699)

U.S. & Int'l
Patents Pending

gelFAST... when you are serious
about hand hygiene.

©2007, Medonyx Inc.
All Rights reserved.
Made in Canada

gelFAST
anywear